HeartSine samaritan® PAD 300P Upgrader (1.4.2/3.2.0)

User instructions

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Windows XP
Windows Vista
Windows 7
Windows 8
Windows 8.1

HeartSine samaritan® PAD 300P Upgrader (1.4.2/3.2.0)

User instructions (U.S. Only)

WARNING - PAD 300P Upgrader is only compatible with 300P. DO NOT use with samaritan® PAD Trainer.

1. Install PAD 300P Upgrader.

EITHER: Download from website:

Go to <http://heartsine.com/recall/system-updates/> & click the "HeartSine samaritan PAD 300P Upgrader (1.4.2 / 3.2.0)" link shown opposite.

Following download, run "PAD300PUprgrader_setup.exe" to install PAD 300P Upgrader.
(Follow on-screen instructions.)

OR: Install from CD:

Insert CD into the computer's CD-ROM drive.
After a few seconds, the 'AutoPlay' window should appear.
Click "Run PAD300PUprgrader_setup.exe" to install PAD 300P Upgrader.
(Follow on-screen instructions.)

*If this does not happen, browse to the CD-ROM and double-click

"PAD300PUprgrader_setup.exe"


PAD 300P Upgrader

(Updated February 05, 2014)

You have been directed to this webpage to enable you to upgrade your PAD300/300P device in accordance with HeartSine Technologies' field safety corrective action dated 11 Sept 2012.

PAD 300P Upgrader User Instructions (PDF, combined English, Deutsch, Español, Français, Italiano)

HeartSine Samaritan PAD 300P Upgrader (1.4.2 / 3.2.0)

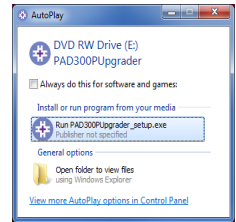2. Start PAD 300P Upgrader.

Start PAD 300P Upgrader by clicking on the icon created on your desktop or press Start menu button and select PAD 300P Upgrader.

After a few seconds the 'PAD 300P Upgrader - (1.4.2/3.2.0)' window will appear.

3. Attach PAD 300P to computer's USB port (see User Manual).

After a few seconds, the samaritan® PAD icon will appear and the connection indicator will turn green.

Click the 'Program PAD' button to begin the upgrade.

Follow any on-screen instructions.

samaritan® PAD icon
connection indicator


4. Following Upgrade.

After successful completion of the device upgrade and successful device on/off check, ensure that the status indicator continues to flash green approximately once every five seconds.

The device can now be returned to use.

If any error messages are heard or if the LED is flashing red you **MUST** contact your distributor or contact support@heartsine.com.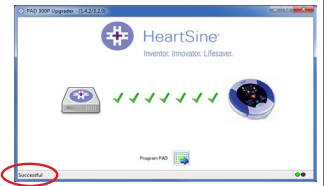

5. Return Proof of Upgrade.

After a successful upgrade, a pdf file will be created on your Desktop. Please return this in one of the following ways:

1. E-mail to support@heartsine.com
2. Print out and post to HeartSine Technologies
3. Fax to 1.215.860.8192

6. Upgrade Issues.

If an error occurs at any point during the upgrade, close the message box and please restart at Step 2.

Important Warning

A small number of users have reported a difficulty with performing the software upgrade. **If you encounter any such issues please contact HeartSine immediately at 1-866-238-2159**

If you have any queries, please contact your distributor or e-mail support@heartsine.com

HeartSine samaritan® PAD 300P Upgrader (1.4.2/3.2.0)

User instructions

1. Install PAD 300P Upgrader.

EITHER: Download from website:

Go to <http://heartsine.com/recall/system-updates/> & click the "HeartSine samaritan PAD 300P Upgrader (1.4.2 / 3.2.0)" link shown opposite.

Following download, run "PAD300PUpgrader_setup.exe" to install PAD 300P Upgrader.
(Follow on-screen instructions.)

OR: Install from CD:

Insert CD into the computer's CD-ROM drive.
After a few seconds, the 'AutoPlay' window should appear.
Click "Run PAD300PUpgrader_setup.exe" to install PAD 300P Upgrader.
(Follow on-screen instructions.)

(If this does not happen, browse to the CD-ROM and double-click "PAD300PUpgrader_setup.exe")


PAD 300P Upgrader

(Updated February 05, 2014)

You have been directed to this webpage to enable you to upgrade your PAD300/300P device in accordance with HeartSine Technologies' field safety corrective action dated 11 Sept 2012.

PAD 300P Upgrader User Instructions (PDF, combined English, Deutsch, Español, Français, Italiano)


HeartSine Samaritan PAD 300P Upgrader (1.4.2 / 3.2.0)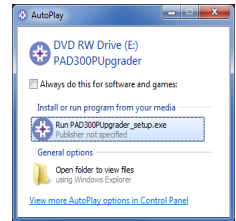

2. Start PAD 300P Upgrader.

Start PAD 300P Upgrader by clicking on the icon created on your desktop, or press Start menu button and select PAD 300P Upgrader.

After a few seconds the 'PAD 300P Upgrader - (1.4.2/3.2.0)' window will appear.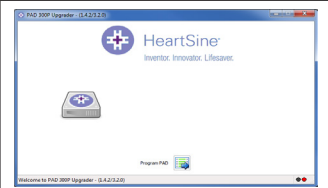


3. Attach PAD 300P to computer's USB port (see User Manual).

After a few seconds, the samaritan® PAD icon will appear and the connection indicator will turn green.

Click the 'Program PAD' button to begin the upgrade.

Follow any on-screen instructions.

samaritan® PAD icon
connection indicator

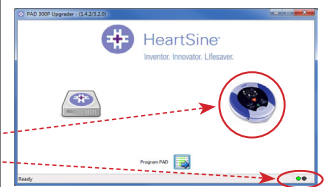4. Following Upgrade.

After successful completion of the device upgrade and successful device on/off check, ensure that the status indicator continues to flash green approximately once every five seconds.

The device can now be returned to use.

If any error messages are heard or if the LED is flashing red you **MUST** contact your distributor or contact support@heartsine.com.

5. Return Proof of Upgrade.

After a successful upgrade, a pdf file will be created on your Desktop. Please return this in one of the following ways:

1. E-mail to support@heartsine.com
2. Print out and post to HeartSine Technologies
3. Fax to +44 (0)28 9093 9401

6. Upgrade Issues.

If an error occurs at any point during the upgrade, close the message box and please restart at Step 2.

Important Warning

A small number of users have reported a difficulty with performing the software upgrade. **If you encounter any such issues please contact HeartSine immediately.**

If you have any queries, please contact your distributor or e-mail support@heartsine.com

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HeartSine Technologies Inc
HeartSine Technologies Ltd