

**stryker**



# HeartSine Gateway™

with LIFELINKcentral™ AED Program Manager

## User Manual



---

# Contents

## Section 1

### **Before you begin** 4

Terminology 4

Symbols 5

## Section 2

### **Introduction to HeartSine Gateway** 6

Indications for use 6

Included in the box 6

Your HeartSine Gateway 7

## Section 3

### **Assemble your HeartSine Gateway** 8

Unpack 8

Disconnect HeartSine Gateway  
from your AED 9

Insert batteries 10

Record your HeartSine Gateway  
11-digit serial number 10

Connect HeartSine Gateway  
to your AED 11

## Section 4

### **Register your HeartSine Gateway** 12

Overview of Stryker AED

Program Management Systems 12

Log in to your account 12

Register your HeartSine Gateway 14

## Section 5

### **Connect HeartSine Gateway to your local network** 17

Setup checklist 17

Connect HeartSine Gateway to the  
network using your mobile app 18

Connect HeartSine Gateway to your  
network using HeartSine Gateway  
Configuration Tool 19

Confirm the connection was successful  
to LIFELINKcentral 26

Connecting your HeartSine Gateway  
to a different Wi-Fi network 27

Connecting your HeartSine Gateway  
to a different HeartSine AED 27

---

<b>Section 6</b>			
<b>Set your Pad-Pak expiration date(s)</b>	<b>28</b>		
<b>Section 7</b>			
<b>Monitoring your HeartSine AED</b>	<b>32</b>		
<b>Section 8</b>			
<b>Caring for your HeartSine Gateway</b>	<b>33</b>		
Maintaining the battery	33		
Cleaning your HeartSine Gateway	34		
Warranty information	34		
<b>Appendix 1</b>			
<b>Warnings and precautions</b>	<b>35</b>		
Warnings	35		
Precautions	36		
<b>Appendix 2</b>			
<b>Troubleshooting</b>	<b>37</b>		
<b>Appendix 3</b>			
<b>LED status indicator</b>	<b>43</b>		
<b>Appendix 4</b>			
<b>LIFELINKcentral AED Program Manager URL and email address</b>	<b>44</b>		
<b>Appendix 5</b>			
<b>Technical data</b>		<b>48</b>	
<b>Appendix 6</b>			
<b>Declaration of Conformity</b>		<b>53</b>	
<b>Warranty</b>		<b>54</b>	

---

## Section 1

# Before you begin

### Terminology

The following terms appear in this manual:

<b>AED</b>	Automated External Defibrillator (your HeartSine samaritan PAD)
<b>MAC Address</b>	Unique identifier assigned to network interfaces
<b>PAD</b>	Public Access Defibrillator
<b>SAM</b>	HeartSine samaritan PAD

## Symbols

The symbols in the following table may be found on HeartSine Gateway, its accessories or packaging.



Manufacturer



HeartSine Gateway is certified to applicable Japanese wireless requirements



Refer to instruction manual



Warning, intentional radiated power from HeartSine Gateway; see Wireless Specifications and local jurisdiction requirements



Do not dispose of this product in the unsorted municipal waste stream; dispose of this product according to local regulations



Recommended storage and operating temperature 0° to 50°C (32° to 122°F)



HeartSine Gateway includes RF transmitter



Product complies with applicable Australian ACMA standards



Mark of conformity to applicable European Directives



DC voltage



Complies with (USA) Federal Communications Commission regulations



Lithium Manganese Dioxide battery



Serial number



Catalog number



Authorized Representative in the European Community

---

## Section 2

# Introduction to HeartSine Gateway

HeartSine Gateway is a self-powered, Wi-Fi based communication module for HeartSine samaritan PAD devices manufactured during or after 2013.

HeartSine Gateway will communicate via Wi-Fi with LIFELINKcentral AED Program Manager enabling AED program managers to readily manage an AED across multiple locations.

This user guide provides instructions on connecting HeartSine Gateway to a compatible AED and to the internet.

---

**NOTE** Your AED will continue to operate as normal with or without HeartSine Gateway attached.

### Indications for use


HeartSine Gateway is indicated for use with a HeartSine samaritan PAD manufactured during or after 2013. (The first two digits in the serial number indicate the year of manufacture. HeartSine Gateway can be used with any HeartSine AED with 13 or above for these two digits.)

### Included in the box

- HeartSine Gateway
- Four CR123A 3V Batteries
- Removal Tool
- Carry Case

## Your HeartSine Gateway

Here is the layout of your HeartSine Gateway.

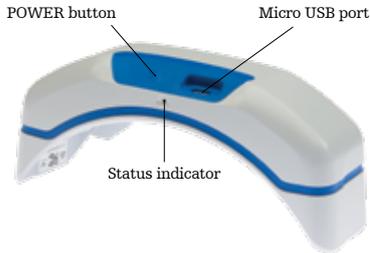

**POWER button:** Press this button to turn on HeartSine Gateway or press and hold the button for six seconds to place HeartSine Gateway into set-up mode.


**Status indicator:** Provides information on the status of HeartSine Gateway.

**Micro USB port:** Enables connection to Saver EVO via a micro USB cable.



Removal tool

**Removal tool:** Used to disconnect HeartSine Gateway from the AED.

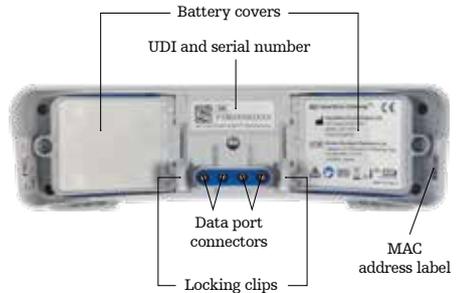

**Battery covers:** Remove to insert or replace the batteries.

**HeartSine Gateway UDI and serial number label:** Provides the Unique Device Identifier (UDI) and Serial Number for HeartSine Gateway.

**Data port connectors:** Connects HeartSine Gateway to the AED.

**Locking clips:** Lock HeartSine Gateway to the AED.

**MAC address label:** Provides the MAC address for HeartSine Gateway.

---

## Section 3

# Assemble your HeartSine Gateway

This section provides the information you need to assemble your HeartSine Gateway and connect it to your AED. If your HeartSine Gateway was provided as part of a HeartSine Connected AED, after you unpack the box you will need to disconnect HeartSine Gateway from the AED and follow the set-up instructions in the AED User Manual before reconnecting HeartSine Gateway.

### **Unpack**

1. Unpack HeartSine Gateway, four CR123A batteries and removal tool.
2. If HeartSine Gateway is attached to an AED, you will need to remove it using the removal tool as shown on page 9.

---

## Disconnect HeartSine Gateway from your AED (if connected)

If HeartSine Gateway is attached to the AED, and you need to change the batteries or for set-up need to view the serial number or MAC address:

1. Insert the removal tool into the back of the AED as shown.
2. Pull HeartSine Gateway away from the AED.



---

**NOTE** If HeartSine Gateway will be stored separately from the AED, insert the blue cover on the AED data port.


---

### **WARNING**

- Use only HeartSine Gateway removal tool provided as using other instruments may damage both HeartSine Gateway and the AED.
  - Place the AED on a flat surface when removing HeartSine Gateway to avoid dropping HeartSine Gateway.
-

## Insert batteries

1. Use an appropriate screwdriver (✳️) to remove the screw on each of the two battery covers.
2. Carefully remove each battery cover.

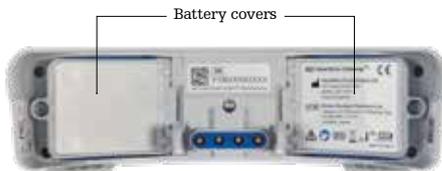

3. Install the four CR123A batteries provided. Ensure the batteries are inserted as indicated in the HeartSine Gateway battery compartment.
4. Replace the battery covers and tighten screws.

### PRECAUTION

If the batteries are not inserted correctly, HeartSine Gateway will not power on.

## Record your HeartSine Gateway serial number

1. Find the HeartSine Gateway 11-digit serial number (where YY = year of manufacture and XXXXXXXX = the unique sequential number) as shown in Section 2 and record it here. You will need this later.



Serial number:

		M								
--	--	---	--	--	--	--	--	--	--	--

## Connect HeartSine Gateway to your AED

1. Remove the blue cover on the data port, located at the top of the AED.



2. Carefully line up HeartSine Gateway with the LED status indicator pointing to the front of the AED.



3. Slide HeartSine Gateway onto the top of the AED allowing the four pins to connect with the contacts on the data port of the AED. When HeartSine Gateway is fully connected, an audible click should be heard as the locking clips snap into place.



4. If a Pad-Pak is not already installed in your AED, be sure and install one before you proceed. Instructions can be found in the AED User Manual.

---

## Section 4

# Register your HeartSine Gateway

### Overview of Stryker AED Program Management Systems

LIFELINKcentral AED Program Manager, an online AED program management system, enables you to manage all your AEDs and accessories, such as a Pad-Pak or Pediatric-Pak. HeartSine Gateway sends data from each AED to LIFELINKcentral AED Program Manager, which is provided with each HeartSine Connected AED/HeartSine Gateway.

### Log in to your account

You should have received a “Welcome to LIFELINKcentral” email with a link to LIFELINKcentral and log on information. Before you attempt to connect your HeartSine Gateway to LIFELINKcentral AED Program Manager for the first time, you should confirm your account is active by logging into LIFELINKcentral.

1. Click the link provided in the email to go to the LIFELINKcentral website.
2. Enter the logon name and password provided in the email.



LIFELINKcentral™  
AED PROGRAM MANAGER

[Forgot your password?](#)[Retrieve/Create Account](#)

---


**NOTE** If you cannot locate this email, check your junk mail folder. If needed refer to Troubleshooting in Appendix 2 of this user manual.

## Register your HeartSine Gateway

When you log into LIFELINKcentral AED Program Manager you must register each HeartSine Gateway using its serial number (located on the underside of HeartSine Gateway as shown in Section 2).

To register each HeartSine Gateway:

1. Click Equipment on the main menu.
2. Point to Manage Equipment.
3. Click Accessories.

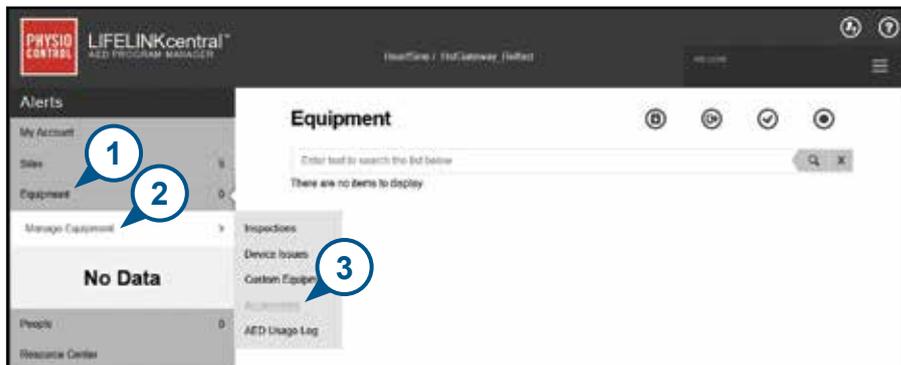

4. From the Accessories screen, click  (Add Accessory).



5. From the pull-down menus, select the site in which HeartSine Gateway will be located, select HeartSine Gateway as the Model, and enter the serial number.

6. Click Submit.

**Add Accessory** X

Site\*  HeartSine / Building 1 (US009680) ▾

Model\* HeartSine Gateway ▾

Location

Serial Number\* 18M00000000

Associated With Device

 Submit Cancel

If successfully registered your HeartSine Gateway will appear in the list of Accessories.

7  
0  
>

## Accessories

Enter text to search the list below

**Not Associated**

**HeartSine Gateway (18M00000000)**

Location	Reception Building 1
Readiness Status	No Data Available

STANDALONE

---

## Section 5

# Connect HeartSine Gateway to your local network

Once you have confirmed your LIFELINKcentral AED Program Manager account is active and you have registered your HeartSine Gateway, it is time to connect HeartSine Gateway to the local network.

To connect HeartSine Gateway to your local network, you can use either the HeartSine Gateway app available in the App store or Google Play, or HeartSine Gateway Configuration Tool from your Wi-Fi enabled computer.



### Setup checklist

Before you can connect your HeartSine Gateway to your chosen Wi-Fi network please check that you have the following:

- Wi-Fi enabled PC or laptop with Windows 7 or above operating system and an internet connection
- Administrator rights for the computer
- Serial number for your HeartSine Gateway
- Location for HeartSine Gateway with a stable and adequate connection to the Wi-Fi network. (Test the signal strength of the Wi-Fi network at the intended location using a different device such as a smartphone.)
- Wi-Fi network name (SSID) and password. If you do not have this information, ask your IT department for assistance.
- Access to your LIFELINKcentral AED Program Manager account.

---

## Connect HeartSine Gateway to your network using the mobile app

1. Download and run “HeartSine Gateway **Configuration Tool**” app from the App Store or Google Play.



2. Follow the instructions provided to:


- Enter the network ID and password
- Press **and hold** the power button **until the solid blue light appears** to put HeartSine Gateway into set-up mode
- **Scan** HeartSine Gateway serial number

When the data you entered for the network has been received by HeartSine Gateway, a green checkmark will appear on the screen and the HeartSine Gateway status indicator will turn solid green. The connection between the app and HeartSine Gateway will be disabled.

3. HeartSine Gateway will attempt to connect the Wi-Fi network. If the connection is successful, the HeartSine Gateway status indicator will be solid green before turning off. If the connection is not successful, the indicator will begin flashing blue. In this case, it is likely the information you entered for the network was not correct and must be re-entered. Confirm that the data you have is correct and repeat each of these steps.
4. For multiple HeartSine Gateway devices, repeat these steps for each HeartSine Gateway.

## Connect HeartSine Gateway to your network using HeartSine Gateway Configuration Tool

You will need to download HeartSine Gateway Configuration Tool from the Resource Center section of LIFELINKcentral AED Program Manager.

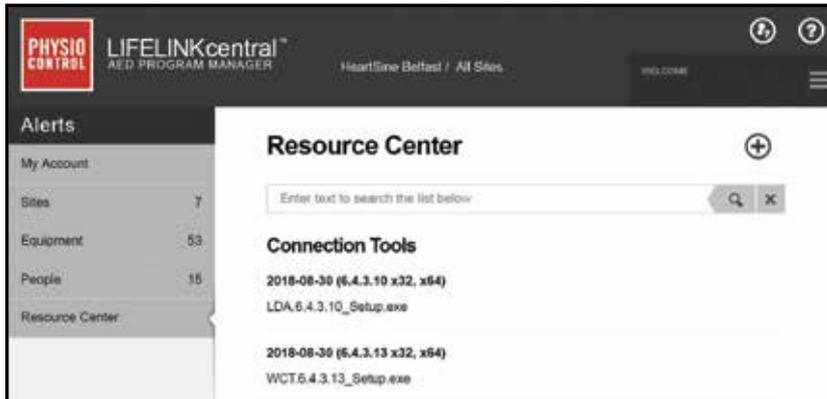

1. Select **RESOURCE CENTER** (in LIFELINKcentral AED Program Manager).
2. Select HeartSine Gateway Configuration Tool.
3. Click  to download HeartSine Gateway Configuration Tool.
4. Once the file is downloaded, double-click it to install. (If you don't see the file, look in your Downloads folder.)  
**PLEASE NOTE:** If a security warning appears, select the option to allow the file.
5. When the **SET-UP WIZARD** appears, follow the onscreen prompts to complete installation.

---

## NOTES

- Depending on your network security settings you may need administration rights to download and install HeartSine Gateway Configuration Tool.
- HeartSine Gateway Configuration Tool should launch automatically after installation. If you need to start HeartSine Gateway Configuration Tool manually, open the START menu on your computer, open the HeartSine Gateway Application folder, and click HeartSine Gateway.
- Your computer may reboot during the installation process. If this happens, the installation will continue automatically.

6. Launch HeartSine Gateway Configuration Tool.



7. Click the right arrow to proceed.

8. As shown on the screen, place HeartSine Gateway into set-up mode by pressing and holding the POWER button for approximately six seconds until the status indicator turns solid blue. You can only connect one HeartSine Gateway at a time. (Further information on the LED status indicator can be found in Appendix 3.)




**NOTE** Your PC / laptop will disconnect from the Wi-Fi network to connect to HeartSine Gateway. When HeartSine Gateway has been configured, you may need to reconnect your PC/laptop to your Wi-Fi network.

9. When the status indicator turns solid blue, click the right arrow to proceed.



10. HeartSine Gateway Configuration Tool will search for all HeartSine Gateways in set-up mode.

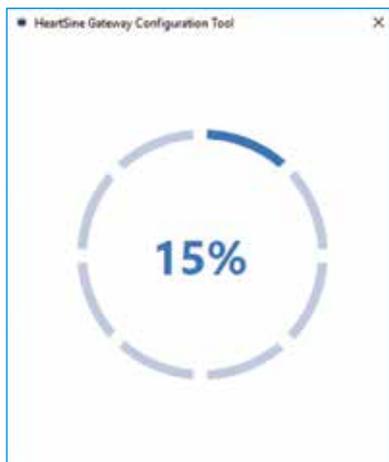

11. When HeartSine Gateway is powered on, in set-up mode and in range of your PC/ laptop, its MAC address and serial number will appear in the drop-down list. To connect a single HeartSine Gateway, select the relevant HeartSine Gateway serial number from this list.



12. Click the right arrow to proceed.

13. From the drop-down list, select your Wi-Fi network and enter the network password. (To view the entered password, hover over the  button.)




14. Click Settings  to view the network type. If you need to change the network type, select an option from the drop-down list .



15. Click the right arrow to proceed.

16. HeartSine Gateway Configuration Tool will try to connect to HeartSine Gateway.

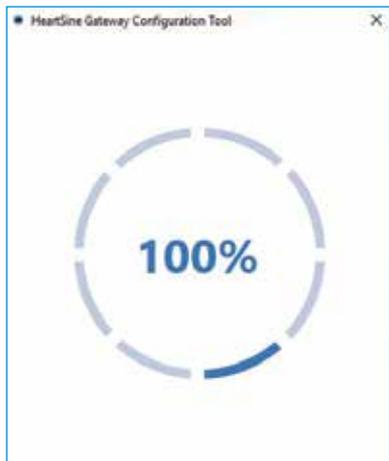

When the data you entered for the network has been received by HeartSine Gateway, a green checkmark will appear on the screen (as shown) and the HeartSine Gateway status indicator will turn solid green. The connection between HeartSine Gateway Configuration Tool and HeartSine Gateway will be disabled.



17. HeartSine Gateway will attempt to connect to the Wi-Fi network. If the connection is successful, the HeartSine Gateway status indicator will be solid green before turning off. If the connection is not successful, the indicator will begin flashing blue. In this case, it is likely the information you entered for the network was not correct and must be re-entered. Confirm that the data you have is correct and repeat each of these steps.
18. For multiple HeartSine Gateway devices, repeat these steps for each HeartSine Gateway.
19. Click the right arrow to close the program.
20. If you need to repeat any steps, relaunch the program and repeat the steps.

**NOTE** In some instances, it may be necessary to provide your IT department with the MAC address of HeartSine Gateway so that it can gain access to your network. The MAC address is provided on the MAC address label as shown in Section 2.

## Confirm the connection was successful to LIFELINKcentral

When you have connected your HeartSine Gateway to your network, go to your LIFELINKcentral AED Program Manager account to confirm your HeartSine Gateway is registered and working. If the connection was successful, the connected HeartSine AED will appear under Equipment in LIFELINKcentral or HeartSine Gateway will appear as an accessory for the AED.

1. Log in to your LIFELINKcentral account.
2. Go to the **EQUIPMENT** page and verify that the status of your AED says **READY**.



## NOTES

- If the status is NOT READY, refer to the Troubleshooting section of this manual.
- If the AED is not shown in the Equipment list, check the HeartSine Gateway readiness status in the Accessories list view. If the readiness status indicates Needs Attention (AED Not Connected), the AED is not properly connected or the AED device model is not supported by HeartSine Gateway.

---

## **Connecting your HeartSine Gateway to a different Wi-Fi network**

To connect your HeartSine Gateway to a different Wi-Fi network, place HeartSine Gateway into set-up mode by holding the POWER button for approximately six seconds (until the status LED on HeartSine Gateway turns solid blue) and repeat the steps to connect to your network.


## **Connecting your HeartSine Gateway to a different HeartSine AED**

To connect your HeartSine Gateway to a different HeartSine AED, use the removal tool to detach HeartSine Gateway from the current AED and attach it to the new AED as described earlier. Press and release the HeartSine Gateway POWER button to automatically connect HeartSine Gateway to its preconfigured Wi-Fi network.



To register each Pad-Pak or Pediatric-Pak:

1. From the Equipment section, click the AED with which the Pad-Pak or Pediatric-Pak will be used and/or stored.
2. Click  to view the accessories associated with the AED.

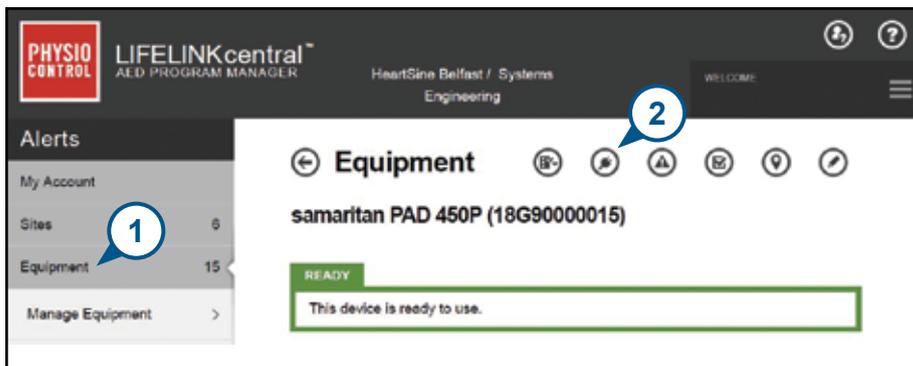

3. From the Accessories screen, click  (Add Accessory).



4. From the Model menu, select Pad-Pak or Pediatric-Pak.




5. Enter the expiration date (month/date/year) provided on the Pad-Pak or Pediatric-Pak label.

6. Enter the Lot Code (number) from the label.

7. If the Pad-Pak or Pediatric-Pak is installed in the AED, click Attached To Device.

8. Click Submit.

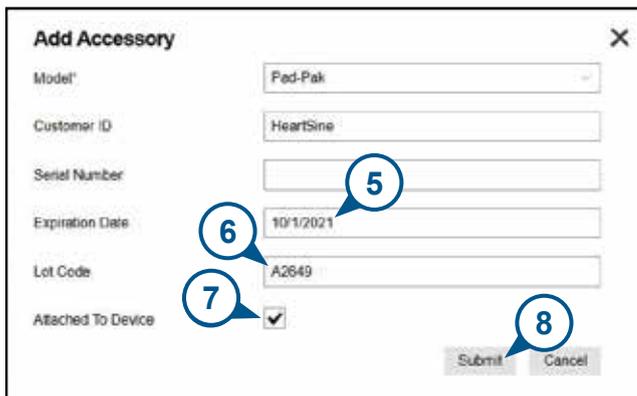

If successfully registered your Pad-Pak or Pediatric-Pak will appear in the list of Accessories.

The screenshot displays the LIFELINKcentral AED PROGRAM MANAGER interface. The top header includes the Physio Control logo, the text "LIFELINKcentral™ AED PROGRAM MANAGER", and the location "Heartline Belfast / Belfast R&D". A "WELCOME" message is visible on the right. A left sidebar contains navigation options: "Alerts", "My Account", "Sites" (7), "Equipment" (24), and "Manage Equipment" (>). The main content area is titled "Accessories" and features a search bar with the placeholder text "Enter text to search the list below". Below the search bar, a specific item is listed: "samaritan PAD 350P (17D90932876)". This item is identified as a "Pad-Pak (A2649)" with a "Location" of "Not Specified" and an "Expiration Date" of "10/1/2021". A green "ATTACHED" button is located at the bottom of the item's details.

---

## Section 7

# Monitoring your HeartSine AED

HeartSine Gateway is designed to operate automatically. HeartSine Gateway will activate and connect to its AED daily, extracting self-test data stored within the AED and will upload the data on a weekly basis to your LIFELINKcentral AED Program Manager account. When HeartSine Gateway has completed the data package upload it will enter a “sleep” mode.

To run an additional “check-in”, press and release the HeartSine Gateway POWER button. This will upload the data from the last AED self-test to your LIFELINKcentral AED Program Manager account.

If there is no communication between HeartSine Gateway and LIFELINKcentral during its scheduled monthly check-in, it is recommended that you visually inspect the AED and connected HeartSine Gateway.

---

 **PRECAUTION** HeartSine Gateway battery life is based on normal operations. Running additional check-ins will deplete the battery life more quickly.

---

---

## Section 8

# Caring for your HeartSine Gateway

### Maintaining the battery

HeartSine Gateway is powered by four CR123A 3V batteries. If HeartSine Gateway LED indicator turns red, the batteries may be low. An email notification will be sent to the pre-set user email account for notification of low battery power on HeartSine Gateway.

Batteries can be replaced by removing HeartSine Gateway from the AED and unscrewing the battery covers (see Section 3).

Replace all batteries at the same time, do not mix new with old.

Dispose of depleted batteries according to local regulations.

---

### **WARNING**

#### SAFETY RISK AND POSSIBLE EQUIPMENT DAMAGE

- Rechargeable batteries **CANNOT** be used in HeartSine Gateway. Using rechargeable batteries will damage HeartSine Gateway and void your warranty.
  - Damaged batteries may leak and cause personal injury or equipment damage. Handle damaged or leaking batteries with extreme care.
-

---

## Cleaning your HeartSine Gateway

It is recommended that you visually inspect your HeartSine Gateway at least once a year. If necessary, clean HeartSine Gateway using a soft cloth dampened by one of the following:

- Nonabrasive soap and water
- Isopropyl alcohol (70% solution)



### PRECAUTION

- **DO NOT** immerse any part of HeartSine Gateway in water or any type of fluid. Immersion in fluids may seriously damage HeartSine Gateway or cause a shock hazard.
- **DO NOT** clean any part of HeartSine Gateway or its accessories with abrasive materials or cleaners (for example bleach or bleach dilutions).
- **DO NOT** attempt to sterilize HeartSine Gateway.

---

## Warranty information

See Appendix 6 for the HeartSine detailed warranty statement.

---

## Appendix 1

# Warnings and precautions

### WARNINGS

#### **Do not open casing**

Do NOT attempt to open or repair HeartSine Gateway under any circumstances. If damage is suspected, contact your local Authorized Distributor or Stryker representative.

#### **Do not use rechargeable batteries**

Do NOT use rechargeable batteries. These batteries will damage HeartSine Gateway and void your warranty.

#### **Removing HeartSine Gateway from the AED**

Do NOT attempt to remove HeartSine Gateway using anything other than the supplied removal tool. Using anything other than the supplied removal tool may damage HeartSine Gateway and the AED.

#### **Micro USB port**

Use only for connection to Saver EVO program. **Do NOT** attempt to use micro USB port for any other purpose.

#### **Faulty HeartSine Gateway**

If you suspect a fault with HeartSine Gateway, remove it from the AED and contact customer support.

#### **EMC warnings**

Use of HeartSine Gateway adjacent to or stacked with other electronic equipment should be avoided because it could result in improper operation. If such use is necessary, HeartSine Gateway and the other equipment should be observed to verify they are operating normally.

---

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of HeartSine Gateway, including cables specified by the manufacturer. Otherwise, degradation of the performance of HeartSine Gateway could result.

## **PRECAUTIONS**

### **Temperature range**

HeartSine Gateway is designed to operate in the temperature range of 0°C to 50°C (32°F to 122°F). Use of HeartSine Gateway outside of this range may cause it to malfunction.

### **Ingress Protection**

HeartSine Gateway has an IP56 rating against dust and sprays of water. Note that the IP56 rating does not cover immersion of HeartSine Gateway in water or any type of fluid. Immersion in fluids may seriously damage HeartSine Gateway or cause fire or a shock hazard.

### **Prolonging Battery Life**

Do not turn on HeartSine Gateway unnecessarily as this may reduce the standby life.

### **Correct Disposal of HeartSine Gateway**

Dispose of HeartSine Gateway in accordance with your national or local regulations or contact your local Authorized Distributor or Stryker representative for assistance.

## Appendix 2

# Troubleshooting

Problem	Solution
<b>HeartSine Gateway will not power on.</b>	<ul style="list-style-type: none"><li>• Ensure batteries have been inserted correctly.</li><li>• Batteries may be depleted; replace the batteries with four new NON-RECHARGEABLE CR123A 3V batteries.</li></ul>
<b>I did not receive an email with my logon name and password.</b>	<ul style="list-style-type: none"><li>• Check your junk mail/ spam folder. You may need to set email filters to allow emails originating from LIFELINKcentral email address in your country. (See Appendix 4 for the list for the correct email address.)</li><li>• Check to see if the email was sent to another person in your organization. If you cannot find the email, contact your local Authorized Distributor or Stryker representative.</li></ul>
<b>HeartSine Gateway will not go into set up mode.</b>	<ul style="list-style-type: none"><li>• Ensure the LED is off then press and hold the POWER button for at least six seconds. The LED turns solid blue. If the LED does not change color, contact your local Authorized Distributor or Stryker representative.</li></ul>
<b>HeartSine Gateway serial number does not appear in HeartSine Gateway Configuration Tool.</b>	<ul style="list-style-type: none"><li>• Ensure HeartSine Gateway is in set-up mode (the LED should be solid blue). If not press and hold the POWER button for at least six seconds. The LED turns solid blue.</li><li>• Ensure HeartSine Gateway is within Wi-Fi range of the PC/laptop.</li><li>• Manually refresh the PC/laptop Wi-Fi list.</li></ul>
<b>HeartSine Gateway is not connecting to HeartSine Gateway Configuration Tool.</b>	<ul style="list-style-type: none"><li>• Ensure the serial number and MAC address are correct.</li><li>• Press and hold the POWER button for six seconds to put HeartSine Gateway into set-up mode (the LED should turn solid blue) and repeat the set-up process.</li></ul>

## Problem

## Solution

### **HeartSine Gateway will not connect to the network.**

- This may mean the Wi-Fi signal strength is not strong enough or available to connect the AED to the Wi-Fi network.
  - Confirm the Wi-Fi is working and has sufficient signal strength.
  - Move the AED closer to your Wi-Fi access point and try again. If you need assistance, contact your local Authorized Distributor or Stryker representative.
- Ensure the SSID and password have been entered correctly.

### **AED is not showing in LIFELINKcentral AED Program Manager.**

- Place HeartSine Gateway into set-up mode by holding the POWER button for six seconds (the LED will turn solid blue) and repeat the steps to connect your HeartSine Gateway. If the problem persists, contact your local Authorized Distributor or Stryker representative for assistance.
- Check the readiness status of HeartSine Gateway in LIFELINKcentral. If the HeartSine Gateway status indicates Needs Attention (AED Not Connected), the AED is not properly connected or the AED device is not supported by HeartSine Gateway. Only HeartSine samaritan devices manufactured during or after 2013 are supported.

### **AED is showing as NOT READY on LIFELINKcentral AED Program Manager.**

- Select the AED from the Equipment section and follow the on-screen advice. If the problem persists, contact your local Authorized Distributor or Stryker representative for assistance.

### **I need my MAC Address.**

- The MAC address label is located on the underside of HeartSine Gateway on the side. (See Section 2 for exact location.)

## Problem

Downloaded HeartSine Gateway Configuration Tool file did not appear on my computer.

If you receive an email, from LIFELINKcentral AED Program Manager, stating that your AED is NOT READY or NEEDS ATTENTION.

## Solution

- Check the 'Downloads' folder on your PC/laptop.

### Follow these steps:

1. Click the link within the email to open LIFELINKcentral and view the equipment status details as shown.

The image shows two screenshots. The left screenshot is an email from LIFELINKcentral. The email header includes: From: LIFELINKcentral AED Status Service; To: LIFELINKcentral AED Status Service (No Reply); Date: 07/24/2018 09:30:57. The main body of the email states: "The readiness status of your device has changed to Not Ready (Battery)." It lists details for a device: Model: HeartFlow Selfless (050002024), Serial Number: 0800000013, AED Location: Building 1, and Readiness Reported: 07/24/18 9:03:55 PM. A blue circle with the number '1' is overlaid on the email text. Below the email is a mobile app screenshot showing the 'Equipment' page for 'Selfless PAC-450P (1800000013)'. A red box highlights a warning message: "This device is not ready to use. Review the battery maintenance. Contact support. Contacted if you search results. If you do not see this message, it may be a temporary issue. Reprogramming needed. go to maintenance to check or place in standby mode." The app also shows a 'Battery of 15' indicator and a 'Manage Equipment' button.

2. Inspect the HeartSine AED and HeartSine Gateway to resolve issue causing the NOT READY or NEEDS ATTENTION status:
  - Check each for visible damage or wear and tear.
  - Press the AED power button to turn it on, then press the power button again to turn it off. Listen for any error prompts and ensure the AED indicator is flashing green.


## Problem

## Solution


(continued)

**If you receive an email, from LIFELINKcentral AED Program Manager, stating that your AED is NOT READY or NEEDS ATTENTION.**

- When you have resolved the issue and the status indicator on the AED is flashing green, return to LIFELINKcentral and select the relevant AED.
- From the Equipment menu, click Inspections .

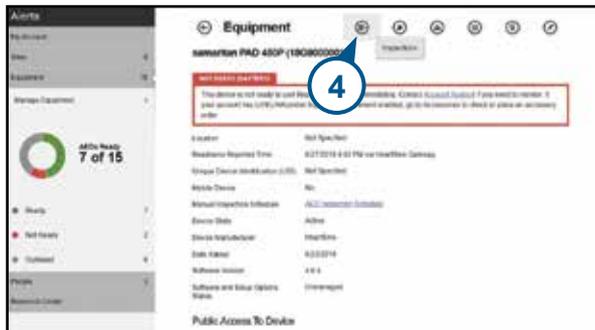

- Click Perform Inspections .

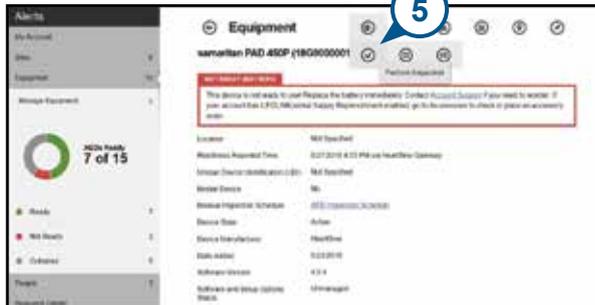

## Problem

## Solution

(continued)

**If you receive an email, from LIFELINKcentral AED Program Manager, stating that your AED is NOT READY or NEEDS ATTENTION.**

- Click the Device Present checkbox (6a) and the Flashing Green button (6b) to indicate that you found the device and resolved the issue.

**Perform Inspection** ✕

saman PAD 450P (18G9000016)

EQUIPMENT INFORMATION ▾

ACCESSORIES ▾

INSPECTION CHECKLIST ▾

Device Present  **6a**

Visible Damage

Readiness Indicator  Flashing Green **6b**

Flashing Red

Off

Readiness Status: Ready

NOTES ▾ **7**

Submit Cancel

- Press Submit.


## Problem

## Solution

(continued)

**If you receive an email, from LIFELINKcentral AED Program Manager, stating that your AED is NOT READY or NEEDS ATTENTION.**

- The AED should now show as READY in LIFELINKcentral (as shown).

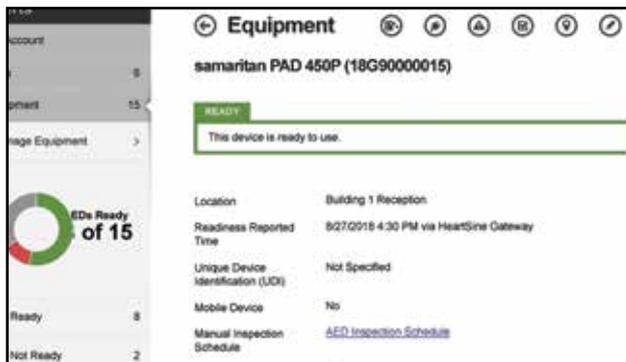

- On HeartSine Gateway attached to the relevant AED, press and release the POWER button.

## Appendix 3

# LED status indicator

The LED status indicator on the front of HeartSine Gateway will indicate its status.

LED Status	Condition	Indication
Solid green 	HeartSine Gateway Active	HeartSine Gateway is active and downloading data from the AED or transmitting data to LIFELINKcentral AED Program Manager.
Solid blue 	Network Setup Mode	System is in Network Setup Mode.
Flashing blue 	Not Connected	HeartSine Gateway is not connected to a network.
Solid red 	Critical Fault	A critical fault is detected. Consult LIFELINKcentral AED Program Manager for details.
Solid black 	Power Off	HeartSine Gateway is powered off.
Solid white 	Not Registered	HeartSine Gateway is not registered in LIFELINKcentral AED Program Manager.

---

## Appendix 4

# LIFELINKcentral AED Program Manager URL and email address

### Network Configuration

Internet access is required for HeartSine Gateway wireless data transmission to work. Network security systems may need to be configured to properly allow communications to the LIFELINKcentral website. There are many methods to secure a network that is connected to the Internet. To help you successfully implement HeartSine Gateway on your network, the following information should be considered.

For all solutions, it may be necessary to know details about where LIFELINKcentral is located on the Internet.

- FODNs for LIFELINKcentral are provided below.
- IP subnet for LIFELINKcentral:  
Canada, Ireland and UK: 74.200.9.46, 255.255.255.240 or a 28 bit submask  
All other locations: 62.29.175.224, 255.255.255.240 or a 28-bit submask
- Please ensure the following IP address is also reachable (pool server):  
74.200.9.47. <https://pool.lifenetsystems.com>

### LIFELINKcentral System Properties

- Protocol - HyperText Transfer Protocol with Privacy (HTTPS)
- Connection - TLS 1.0, 1.1, 1.2; 128 bit or higher encryption; RSA 2048 bit and ECDSA 256 bits key exchange

---

## **Email Filters**

Email filters may need to be set to allow emails originating from LIFELINKcentral. The email for your country is provided in the table on pages 46 and 47.

## **Supported Security Protocols**

- Open\*
- WPA-Personal
- WPA2-Personal

## **Non-Supported Security Protocols**

- WPA-Enterprise
- WPA2-Enterprise
- WEP

\*Open networks without a user agreement page are supported.

Country	URL Record Name	Email address*
Argentina	<a href="https://lifelink-ar.lifenetsystems.com">https://lifelink-ar.lifenetsystems.com</a>	<a href="mailto:web@lifelink-ar.lifenetsystems.com">web@lifelink-ar.lifenetsystems.com</a>
Australia	<a href="https://lifelink-au.lifenetsystems.com">https://lifelink-au.lifenetsystems.com</a>	<a href="mailto:web@lifelink-au.lifenetsystems.com">web@lifelink-au.lifenetsystems.com</a>
Austria	<a href="https://lifelink-at.lifenetsystems.com">https://lifelink-at.lifenetsystems.com</a>	<a href="mailto:web@lifelink-at.lifenetsystems.com">web@lifelink-at.lifenetsystems.com</a>
Bahrain	<a href="https://lifelink-bh.lifenetsystems.com">https://lifelink-bh.lifenetsystems.com</a>	<a href="mailto:web@lifelink-bh.lifenetsystems.com">web@lifelink-bh.lifenetsystems.com</a>
Belgium	<a href="https://lifelink-be.lifenetsystems.com">https://lifelink-be.lifenetsystems.com</a>	<a href="mailto:web@lifelink-be.lifenetsystems.com">web@lifelink-be.lifenetsystems.com</a>
Canada	<a href="https://lifelink-ca.lifenetsystems.com">https://lifelink-ca.lifenetsystems.com</a>	<a href="mailto:web@lifelink-ca.lifenetsystems.com">web@lifelink-ca.lifenetsystems.com</a>
Chile	<a href="https://lifelink-cl.lifenetsystems.com">https://lifelink-cl.lifenetsystems.com</a>	<a href="mailto:web@lifelink-cl.lifenetsystems.com">web@lifelink-cl.lifenetsystems.com</a>
Costa Rica	<a href="https://lifelink-cr.lifenetsystems.com">https://lifelink-cr.lifenetsystems.com</a>	<a href="mailto:web@cr-lifenetsystems.com">web@cr-lifenetsystems.com</a>
Denmark	<a href="https://lifelink-dk.lifenetsystems.com">https://lifelink-dk.lifenetsystems.com</a>	<a href="mailto:web@lifelink-dk.lifenetsystems.com">web@lifelink-dk.lifenetsystems.com</a>
Faroe Islands	<a href="https://lifelink-fo.lifenetsystems.com">https://lifelink-fo.lifenetsystems.com</a>	<a href="mailto:web@lifelink-fo.lifenetsystems.com">web@lifelink-fo.lifenetsystems.com</a>
Finland	<a href="https://lifelink-fi.lifenetsystems.com">https://lifelink-fi.lifenetsystems.com</a>	<a href="mailto:web@lifelink-fi.lifenetsystems.com">web@lifelink-fi.lifenetsystems.com</a>
France	<a href="https://lifelink-fr.lifenetsystems.com">https://lifelink-fr.lifenetsystems.com</a>	<a href="mailto:web@lifelink-fr.lifenetsystems.com">web@lifelink-fr.lifenetsystems.com</a>
Germany	<a href="https://lifelink-de.lifenetsystems.com">https://lifelink-de.lifenetsystems.com</a>	<a href="mailto:web@lifelink-de.lifenetsystems.com">web@lifelink-de.lifenetsystems.com</a>
Greenland	<a href="https://lifelink-gl.lifenetsystems.com">https://lifelink-gl.lifenetsystems.com</a>	<a href="mailto:web@lifelink-gl.lifenetsystems.com">web@lifelink-gl.lifenetsystems.com</a>
Hong Kong	<a href="https://lifelink-hk.lifenetsystems.com">https://lifelink-hk.lifenetsystems.com</a>	<a href="mailto:web@lifelink-hk.lifenetsystems.com">web@lifelink-hk.lifenetsystems.com</a>
Ireland	<a href="https://lifelink-ie.lifenetsystems.com">https://lifelink-ie.lifenetsystems.com</a>	<a href="mailto:web@lifelink-ie.lifenetsystems.com">web@lifelink-ie.lifenetsystems.com</a>
Israel	<a href="https://lifelink-il.lifenetsystems.com">https://lifelink-il.lifenetsystems.com</a>	<a href="mailto:web@lifelink-il.lifenetsystems.com">web@lifelink-il.lifenetsystems.com</a>
Italy	<a href="https://lifelink-it.lifenetsystems.com">https://lifelink-it.lifenetsystems.com</a>	<a href="mailto:web@lifelink-it.lifenetsystems.com">web@lifelink-it.lifenetsystems.com</a>
Japan	<a href="https://lifelink-jp.lifenetsystems.com">https://lifelink-jp.lifenetsystems.com</a>	<a href="mailto:web@jp-lifenetsystems.com">web@jp-lifenetsystems.com</a>
Jordan	<a href="https://lifelink-jo.lifenetsystems.com">https://lifelink-jo.lifenetsystems.com</a>	<a href="mailto:web@lifelink-jo.lifenetsystems.com">web@lifelink-jo.lifenetsystems.com</a>

Country	URL Record Name	Email address*
Kuwait	<a href="https://lifelink-kw.lifenetsystems.com">https://lifelink-kw.lifenetsystems.com</a>	<a href="mailto:web@lifelink-kw.lifenetsystems.com">web@lifelink-kw.lifenetsystems.com</a>
Liechtenstein	<a href="https://lifelink-li.lifenetsystems.com">https://lifelink-li.lifenetsystems.com</a>	<a href="mailto:web@lifelink-li.lifenetsystems.com">web@lifelink-li.lifenetsystems.com</a>
Luxembourg	<a href="https://lifelink-lu.lifenetsystems.com">https://lifelink-lu.lifenetsystems.com</a>	<a href="mailto:web@lifelink-lu.lifenetsystems.com">web@lifelink-lu.lifenetsystems.com</a>
Malta	<a href="https://lifelink-mt.lifenetsystems.com">https://lifelink-mt.lifenetsystems.com</a>	<a href="mailto:web@lifelink-mt.lifenetsystems.com">web@lifelink-mt.lifenetsystems.com</a>
Mexico	<a href="https://lifelink-mx.lifenetsystems.com">https://lifelink-mx.lifenetsystems.com</a>	<a href="mailto:web@lifelink-mx.lifenetsystems.com">web@lifelink-mx.lifenetsystems.com</a>
Netherlands	<a href="https://lifelink-nl.lifenetsystems.com">https://lifelink-nl.lifenetsystems.com</a>	<a href="mailto:web@lifelink-nl.lifenetsystems.com">web@lifelink-nl.lifenetsystems.com</a>
New Zealand	<a href="https://lifelink-nz.lifenetsystems.com">https://lifelink-nz.lifenetsystems.com</a>	<a href="mailto:web@lifelink-nz.lifenetsystems.com">web@lifelink-nz.lifenetsystems.com</a>
Norway	<a href="https://lifelink-no.lifenetsystems.com">https://lifelink-no.lifenetsystems.com</a>	<a href="mailto:web@lifelink-no.lifenetsystems.com">web@lifelink-no.lifenetsystems.com</a>
Oman	<a href="https://lifelink-om.lifenetsystems.com">https://lifelink-om.lifenetsystems.com</a>	<a href="mailto:web@lifelink-om.lifenetsystems.com">web@lifelink-om.lifenetsystems.com</a>
Poland	<a href="https://lifelink-pl.lifenetsystems.com">https://lifelink-pl.lifenetsystems.com</a>	<a href="mailto:web@pl-lifenetsystems.com">web@pl-lifenetsystems.com</a>
Qatar	<a href="https://lifelink-qa.lifenetsystems.com">https://lifelink-qa.lifenetsystems.com</a>	<a href="mailto:web@lifelink-qa.lifenetsystems.com">web@lifelink-qa.lifenetsystems.com</a>
Saudi Arabia	<a href="https://lifelink-sa.lifenetsystems.com">https://lifelink-sa.lifenetsystems.com</a>	<a href="mailto:web@lifelink-sa.lifenetsystems.com">web@lifelink-sa.lifenetsystems.com</a>
Singapore	<a href="https://lifelink-sg.lifenetsystems.com">https://lifelink-sg.lifenetsystems.com</a>	<a href="mailto:web@lifelink-sg.lifenetsystems.com">web@lifelink-sg.lifenetsystems.com</a>
South Africa	<a href="https://lifelink-za.lifenetsystems.com">https://lifelink-za.lifenetsystems.com</a>	<a href="mailto:web@lifelink-za.lifenetsystems.com">web@lifelink-za.lifenetsystems.com</a>
Spain	<a href="https://lifelink-es.lifenetsystems.com">https://lifelink-es.lifenetsystems.com</a>	<a href="mailto:web@lifelink-es.lifenetsystems.com">web@lifelink-es.lifenetsystems.com</a>
Sweden	<a href="https://lifelink-se.lifenetsystems.com">https://lifelink-se.lifenetsystems.com</a>	<a href="mailto:web@lifelink-se.lifenetsystems.com">web@lifelink-se.lifenetsystems.com</a>
Switzerland	<a href="https://lifelink-ch.lifenetsystems.com">https://lifelink-ch.lifenetsystems.com</a>	<a href="mailto:web@lifelink-ch.lifenetsystems.com">web@lifelink-ch.lifenetsystems.com</a>
United Arab Emirates	<a href="https://lifelink-ae.lifenetsystems.com">https://lifelink-ae.lifenetsystems.com</a>	<a href="mailto:web@lifelink-ae.lifenetsystems.com">web@lifelink-ae.lifenetsystems.com</a>
United Kingdom	<a href="https://lifelink-uk.lifenetsystems.com">https://lifelink-uk.lifenetsystems.com</a>	<a href="mailto:web@uk-lifenetsystems.com">web@uk-lifenetsystems.com</a>
United States	<a href="https://lifelink.lifenetsystems.com">https://lifelink.lifenetsystems.com</a>	<a href="mailto:no-reply@lifelink.lifenetsystems.com">no-reply@lifelink.lifenetsystems.com</a>

\* These email addresses are unmonitored and will not accept incoming emails.

---

## Appendix 5

# Technical data

### Physical specifications

---

Size	17 x 7 x 5 cm (6.69 x 2.76 x 1.97 in)
------	---------------------------------------

---

Weight	115g (0.25 lb)
--------	----------------

---

Weight (with batteries installed)	185g (0.41 lb)
--------------------------------------	----------------

---

### Environmental specifications

---

Operating/Standby Temperature	0°C to 50°C (32°F to 122°F)
----------------------------------	-----------------------------

---

Transport Temperature	-10°C to 50°C (14°F to 122°F) for up to two days. If the device has been stored below 0°C (32°F), it should be returned to an ambient temperature between 0°C to 50°C (32°F to 122°F) for at least 24 hours before use.
-----------------------	---

---

Relative Humidity	5% to 95% non-condensing
-------------------	--------------------------

---

Enclosure	IEC 60529 IP56
-----------	----------------

---

Altitude	0 to 4572 meters (0 to 15,000 feet)
----------	-------------------------------------

---

Shock	MIL-STD 810F: 2000 Method 516.5 Procedure 1
-------	---

---

Vibration	MIL-STD 810F: 2000 Method 514.5 Procedure 1 categories 4 & 7
-----------	--

---

EMC	IEC 60601-1-2
-----	---------------

---

---

## Communications

Communications	Wireless 802.11 b/g/n
Frequency Bands	2412-2472 Mhz
Maximum Frequency Power Transmitted	18dBm

## Batteries

Type	CR123A 3V, Non-Rechargeable
Type Number	6205
Designation IEC	CR 17345
Size	34.5 x 17 mm (1.35 x 0.67 in)
Weight	17g (0.04 lb) (1 x Battery) 68g (0.15 lb) (4 x Batteries)
System	Lithium-Manganese Dioxide (LiMnO <sub>2</sub> ) / Organic Electrolyte
UL Recognition	MH 13654 (N)
Nominal Voltage	3 V
Typical Capacity C Load 100 Ohm, at 20°C	1550 mAh down to 2 V
Volume	7 ccm (0.43 in <sup>3</sup> )
Coding	Date of Manufacturing Month / Year

## Electromagnetic conformity

HeartSine Gateway is suitable for use in all professional and domestic establishments. It is not intended for use near intentional transmitters of radio energy such as high frequency surgical equipment, radar installations or radio transmitters, nor in the vicinity of magnetic resonance imaging (MRI) equipment.

HeartSine Gateway is intended for use in the electromagnetic environments specified in the tables below. The user of HeartSine Gateway should assure that it is used in such an environment.

The essential performance of HeartSine Gateway is the ability to periodically request and receive data via the existing USB interface of the host defibrillator, and to wirelessly transmit this data over an established Wi-Fi connection. Operation outside of the environments specified below may result in improper operation of HeartSine Gateway device.

There are no special maintenance procedures required to ensure that the essential performance and basic safety of HeartSine Gateway are maintained with regard to electromagnetic disturbances over the service life of the device.

Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF CISPR 11	Group 1 Class B	HeartSine Gateway uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
Harmonic Emission IEC/EN 61000-3-2	Not Applicable	
Voltage Fluctuations/ Flicker Emission IEC/EN 61000-3-3	Not Applicable	HeartSine Gateway is suitable for use in all establishments, including domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

<b>Immunity Test</b>	<b>IEC 60601 Test Level</b>	<b>Compliance Level</b>	<b>Electromagnetic Environment – Guidance</b>
Electrostatic Discharge (ESD) IEC/EN 61000-4-2	± 8kV Contact ± 15kV Air	± 8kV Contact ± 15kV Air	There are no special requirements with respect to electrostatic discharge.
Electrical fast transients/bursts IEC/EN 61000-4-4	Not Applicable	Not Applicable	
Surges, line to line IEC/EN 61000-4-5	Not Applicable	Not Applicable	
Surges, line to ground IEC/EN 61000-4-5	Not Applicable	Not Applicable	
Voltage dips, interruptions and variations on power supply input lines IEC/EN 61000-4-11	Not Applicable	Not Applicable	
Power Frequency (50/60Hz) Magnetic Field IEC/EN 61000-4-8	30A/m	30A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.  There are no special requirements for non-commercial/non-hospital environments.
Radiated RF IEC/EN 61000-4-3	10 V/m 80MHz – 2.7GHz	10V/m 80MHz – 2.7GHz	Portable and mobile RF communications equipment should be used no closer to any part of the HeartSine samaritan PAD, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter, or 30cm, whichever is greater. <sup>a</sup>  Interference may occur in the vicinity of equipment marked with this symbol. 
Conducted RF IEC/EN 61000-4-6	Not Applicable	Not Applicable	

---

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for cellular telephones, amateur radio, FM and AM radio broadcast and television broadcast cannot be predicted theoretically with a great deal of accuracy. In such cases, an electromagnetic site survey should be considered to properly assess the electromagnetic environment. If the measured field strength in the location in which HeartSine Gateway is intended to be used exceeds the applicable RF compliance levels noted above, HeartSine Gateway should be observed to verify normal operation. If abnormal performance is observed, consideration should be given to relocating HeartSine Gateway if possible.

---

**NOTE** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## **Federal Communications Commission (FCC) declaration**

HeartSine Gateway is capable of transferring data records by wired or wireless connection.

HeartSine Gateway complies with Part 15 of the FCC rules and with Licence exempt RSS standards of Industry Canada.

Operation is subject to the following two conditions:

1. HeartSine Gateway may not cause harmful interference, and
2. HeartSine Gateway must accept any interference received, including interference that may cause undesired operation.

---

 **PRECAUTION** Changes or modifications not expressly approved by HeartSine could void the user's authority to operate the equipment. The term "IC:" before the radio certification number only signifies that Industry Canada technical specifications were met.

---

# Appendix 6

## Declaration and warranty

### EU Declaration of Conformity

Radio equipment:

Name: **HeartSine Gateway**  
Model: **Gateway**  
Catalog: ACC-GTW-XX-01, 350-STR-XX-GW, 360-STR-XX-GW, 450-STR-XX-GW, 500-STR-XX-GW (where XX denotes language)

Manufacturer:

Name: **HeartSine Technologies Ltd.**  
Address: **203 Airport Road West, Belfast, BT3 9ED**  
Country: **United Kingdom**

This declaration of conformity is issued under the sole responsibility of HeartSine Technologies Ltd. who declares that the above named HeartSine Gateway is in conformity with the following regulations:

Radio Equipment Directive (RED)	2014/53/EU
ROHS2 Directive 2 (Article 4)	2011/65 EU with exceptions Annex III (6c – lead in copper alloy)

Conformity to the essential requirements of the legislation(s) have been demonstrated by using the following standards:

**Health and Safety Article 3.1a:** IEC 60601-1:2005

**EMC Article 3.1b:** Directive 2014/30/EU, EN55032 / EN55011:2016, EN60601-1-2:2014, EN 301 489-1 V2.1.1, EN 301 489-17 V3.1.1, EN61000-4-2:2009, EN61000-4-3:2006, EN61000-4-8:2010

**Spectrum Article 3.2:** EN 300 328 v2.1.1

Signed for and on behalf of: HeartSine Technologies Ltd  
Place and Date of Issue: Belfast, United Kingdom, 8th November 2019

Signature  \_\_\_\_\_ Date 09 Jan 2020 \_\_\_\_\_

Rebecca Funston  
Senior Manager, Regulatory Affairs & Quality Assurance  
HeartSine Technologies Ltd.

---

## Warranty

**What is covered?** HeartSine provides to the original end user a limited warranty that all HeartSine products that are purchased from a distributor, sub-distributor, person or entity authorized by HeartSine Technologies, Ltd. ("Authorized Agents") are substantially free from defects in material and workmanship. This limited warranty applies only to the original end user and may not be assigned or transferred. An original end user is one who is able to provide proof of purchase from HeartSine or an Authorized Agent. Persons who are not original end users take the products "as is" and with all faults. Please be prepared to provide proof of purchase demonstrating that you are the original end user and eligible to make a valid claim under this warranty. If you are not sure if the distributor, sub-distributor, person or entity from whom you purchased any HeartSine products is authorized by HeartSine Technologies, Ltd. please contact Customer Support on +44 28 9093 9400 or heartsinesupport@stryker.com.

**For how long?** HeartSine warrants, from the date of the sale to the original end user, the HeartSine samaritan PAD for the full eight (8) year service life and the HeartSine samaritan Trainer and HeartSine Gateway for a period of two (2) years. Products with a stated expiration date are warranted until such expiration date.

**Limited warranty does not cover:** This limited warranty does not cover defects or damages of any sort resulting from, but not limited to, accidents, damage while in transit to our service location, alterations, unauthorized service, unauthorized product case opening, failure to follow instructions, improper use, improper or inadequate maintenance, abuse, neglect, fire, flood, war or acts of God. We do not warrant your HeartSine products to be compatible with any other medical devices.

**This limited warranty is void if:** You purchased any HeartSine products from anyone other than an Authorized Agent; your HeartSine product is serviced or repaired by anyone other than HeartSine Technologies, Ltd.; your HeartSine product is opened by unauthorized personnel or if a product is not used in accordance with the Instructions for Use and the Indications for Use provided with your product; your HeartSine product is used in conjunction with incompatible parts or accessories, including, but not limited to batteries. Parts and accessories are not compatible if they are not HeartSine products.

**What you should do:** As the original end user you should send the completed Warranty Card within 30 days of original purchase to:

HeartSine Technologies, Ltd.  
203 Airport Road West  
Belfast, BT3 9ED, United Kingdom

or register online using the Warranty Registration link on our website [www.heartsine.com](http://www.heartsine.com). To obtain warranty service for your HeartSine product, contact your local HeartSine Authorized Agent or call Customer Support on +44 28 9093 9400. Our technical representative will try to resolve your issue over the phone. If necessary, and at our sole discretion, we will arrange for service or a replacement of your HeartSine product. You must not send back any product without our authorization.

**What we will do:** If your HeartSine product contains defects in material or workmanship and it is returned, at the direction of a technical service representative, within the warranty period, we, at our sole discretion, will repair your product or replace it with a new or reconditioned product of the same or similar design. The repaired or reconditioned product will be warranted subject to the terms and conditions of this limited warranty for either (a) 90 days or

---

(b) the remainder of the original warranty period, whichever is longer, provided the warranty applies and the warranty period has not expired.

If our inspection does not detect any defects in material or workmanship of your HeartSine product, regular service charges will apply.

**Obligations and limitation of liability:**

THE FOREGOING LIMITED WARRANTY IS IN LIEU OF AND SPECIFICALLY EXCLUDES AND REPLACES ALL OTHER EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, TITLE, AND NON-INFRINGEMENT. Some states do not allow limitations on how long an implied warranty lasts, so this limitation may not apply to you.

NO PERSON (INCLUDING ANY AGENT, DEALER, OR REPRESENTATIVE OF HEARTSINE TECHNOLOGIES,

LTD.) IS AUTHORIZED TO MAKE ANY REPRESENTATION OR WARRANTY CONCERNING HEARTSINE PRODUCTS, EXCEPT TO REFER TO THIS LIMITED WARRANTY.

YOUR EXCLUSIVE REMEDY WITH RESPECT TO ANY AND ALL LOSSES OR DAMAGES RESULTING FROM ANY CAUSE WHATSOEVER SHALL BE AS SPECIFIED ABOVE. HEARTSINE TECHNOLOGIES, LTD. SHALL IN NO EVENT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES OF ANY KIND, INCLUDING, BUT NOT LIMITED TO, EXEMPLARY DAMAGES, PUNITIVE DAMAGES, COMMERCIAL LOSS FROM ANY CAUSE, BUSINESS INTERRUPTION OF ANY NATURE, LOSS OF PROFITS OR PERSONAL INJURY OR DEATH EVEN IF WE HAVE BEEN ADVISED OF THE POSSIBILITIES OF SUCH DAMAGES, HOWEVER OCCASIONED, WHETHER BY NEGLIGENCE OR OTHERWISE. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

Stryker Corporation or its affiliates own, use, or have applied for the following trademarks or service marks: HeartSine, samaritan, Pad-Pak, Pediatric-Pak, Saver EVO, LIFELINKcentral, LIFENET and Stryker. All other trademarks are trademarks of their respective owners or holders.

HeartSine Gateway user manual can be found on our website at  
<https://heartsine.com/support/product-manuals/>



For further information contact us at **heartsinesupport@stryker.com** or visit our website at **heartsine.com**.



HeartSine Technologies, Ltd.  
203 Airport Road West  
Belfast, BT3 9ED  
United Kingdom  
Tel +44 28 9093 9400  
Fax +44 28 9093 9401

#### **U.S./Latin America Customer Support**

11811 Willows Road NE  
Redmond, WA, 98052 U.S.A.  
Toll free 866 478 7463  
Tel +1 215 860 8100  
Fax +1 215 860 8192



© 2019 HeartSine Technologies. All rights reserved.  
Made in U.K. Date of Issue: January 2020.  
HeartSine Gateway is not available in all countries.  
H052-019-300-1