

HeartSine Gateway™

1x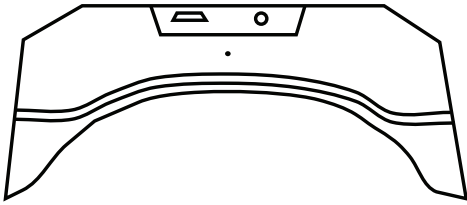

1x



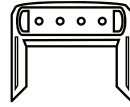
4x



Set-up


1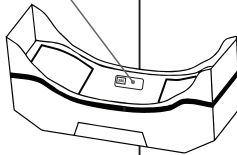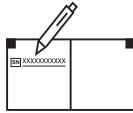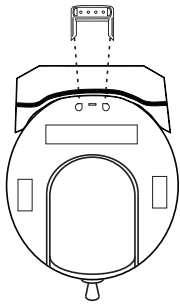


2

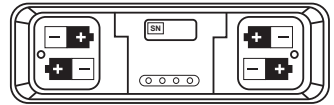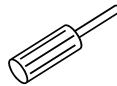3


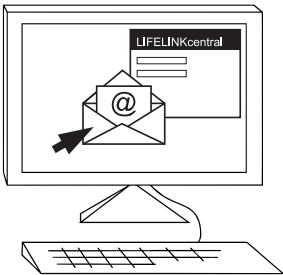
4

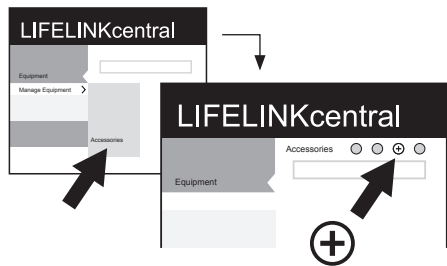5

6

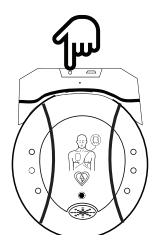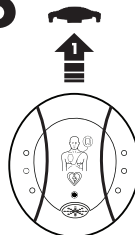HeartSine Gateway is a self-powered, Wi-Fi® communication module for HeartSine® samaritan® PAD devices manufactured during or after 2013.

Set-up

1. If required, remove the HeartSine Gateway from the HeartSine samaritan PAD using the removal tool provided.

Only use removal tool to detach HeartSine Gateway from HeartSine samaritan PAD.

Record the 11-digit HeartSine Gateway serial number here:

SN Y Y _ _ _ _ _

2. Unscrew battery covers and insert all 4 batteries in the correct orientation. Replace the battery covers.

Do not use rechargeable batteries.

3. Click the link provided by email to log into LIFELINKcentral™ AED Program Manager to set up and register your HeartSine Gateway.
4. To first register your HeartSine Gateway within LIFELINKcentral, click Equipment > Manage Equipment > Accessories >  > Model (HeartSine Gateway) and enter HeartSine Gateway serial number.
5. To connect HeartSine Gateway to your local Wi-Fi network, download and run HeartSine Gateway Configuration Tool from LIFELINKcentral Resource Center, or run the HeartSine Gateway app from the App Store or Google Play (using the QR codes provided in Step 5).

Ensure other portable RF communications equipment is no closer than 30 cm (12 in). This could result in degradation of the performance of the HeartSine Gateway.

6. (If required, remove data port cover from HeartSine samaritan PAD). Attach HeartSine Gateway to the data port of your AED. Turn HeartSine Gateway on and off to link it to the attached HeartSine samaritan PAD. Confirm HeartSine Gateway and samaritan PAD appear in LIFELINKcentral by checking the Equipment section of LIFELINKcentral.

Use the QR code on top front cover to download the HeartSine Gateway User Manual from heartsine.com/heartsine-gateway/ for instructions to set the expiration dates for each Pad-Pak™ and/or Pediatric-Pak™. A paper version of the user manual is available on request.

Frequency specifications

Hereby, HeartSine Technologies Ltd. declares that the HeartSine Gateway is in compliance with Directive 2014/53/EU.

Frequency bands: 2412-2472 megahertz (MHz)

Radio frequency power: No greater than 20 dbm

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HeartSine Technologies Ltd., HeartSine Gateway'in 2014/53/AB sayılı Direktife uygun olduğunu beyan eder.
HeartSine Technologies, Ltd.
Redmond, WA, 98052 U.S.A.