


HeartSine® samaritan® PAD

Trainer family

Training automated external defibrillator (AED)
with remote control

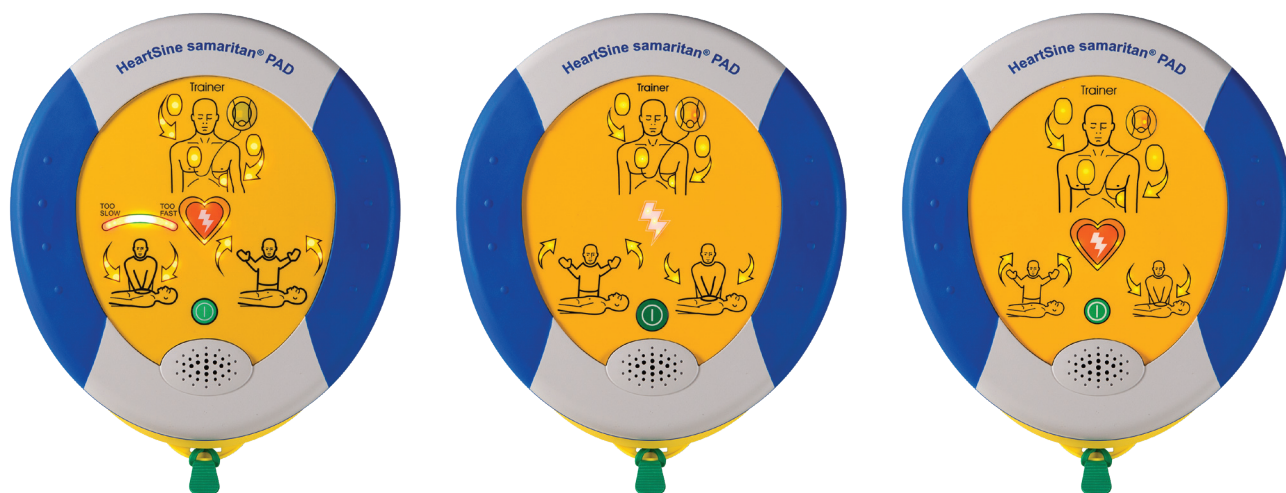

Realistic training demonstration

The easy-to-use HeartSine samaritan PAD Trainer with remote control guides users through simulated analysis, energy delivery and prompted cardiopulmonary resuscitation (CPR) with the look and feel of a live HeartSine samaritan public access defibrillator (PAD) without the actual charge and discharge of an electrical shock.

The HeartSine samaritan PAD Trainer is available in three models: SAM 450P, SAM 360P and SAM 350P. The SAM 450P Trainer simulates the SAM 450P AED by providing the same visual and verbal CPR Rate Advisor prompts.



Realistic

Voice and visual prompts, CPR Rate Advisor and CPR coaching match those of the corresponding HeartSine samaritan PAD defibrillator. Likewise, size and shape are the same, but unique labeling, membrane color and Pad-Pak color are designed to prevent inadvertent use in an actual emergency.

Simulate the full range of “real life” scenarios

The multi-function remote control enables you to select among six AHA/ERC scenarios for live demonstrations or training. Additionally, the SAM 450P Trainer remote control provides the training instructor with the flexibility to choose the appropriate CPR Rate Advisor prompts based on the student’s CPR compression rate.

Each Trainer is operated by a specific remote control, enabling multiple devices to be used in close proximity without interference. The remote control will operate with the Trainer within a range of 3 to 5 meters (10 to 15 feet).

Appropriate for all skill levels

The HeartSine Trainer can be used by businesses to train employee first responders on the use of the HeartSine samaritan PAD. In addition, it meets the needs of professional trainers, either teaching first-time lay users or providing refresher training for experienced emergency responders.

Cost-effective

The HeartSine Trainer is powered by a rechargeable battery. Electrode pads are cost-effective and reusable, and mimic the Pad-Pak battery/pad cartridge unique to the HeartSine samaritan PAD defibrillator.



In the box

The HeartSine samaritan PAD Trainer package includes:

- HeartSine samaritan PAD Trainer
- Trainer-Pak
- Three pairs of training electrode pads
- Remote control with AAA batteries
- Soft carry case with two zipper compartments
- Battery charger
- User manual

Scenarios

The HeartSine samaritan PAD Trainer integrates realistic rescue scenarios for AED training and adheres to the latest AHA/ERC guidelines of one shock followed by two minutes of CPR. A metronome and flashing prompt at 110 beats per minute help to establish correct rate for chest compressions.

With the SAM 450P Trainer (TRN-450), the training instructor can observe the CPR being performed by the students, and use the remote control to provide feedback on rate of compressions. Depending on the remote button pushed, the student may hear “Push faster,” “Push slower” or “Good speed”. If the student does not begin CPR when prompted by the Trainer, the training instructor can push a button to prompt the student to “Begin CPR”. As on the SAM 450P, visual guides will accompany the verbal prompt to assist the student to perform the appropriate rate of compressions.

The following are selectable training scenarios. The training instructor can manually select and play the CPR Rate Advisor verbal and visual prompts at any time while CPR is being performed.

- **Shock advised (SA)**
Shock Advised condition, followed by CPR
- **No shock advised (NSA)**
No Shock Advised condition, followed by CPR
- **Check pads**
Check pads followed by SA condition and CPR
- **Analyzing mode**
Analyzing mode followed by SA and CPR, SA and CPR, and NSA and CPR
- **Shock advised**
Shock Advised condition, followed by CPR (compressions and breaths)
- **No shock advised**
No Shock Advised condition, followed by CPR (compressions and breaths)

Features

Selectable training scenarios

Six pre-programmed AHA/ERC scenarios.

Training pads

Three pairs of reusable, replaceable training electrode pads are included. Compatible with most training manikins, the electrode pads are easy to place back onto the pad liner.

Cable rewind mechanism

Easy replacement of pads and cables after each session reduces time spent resetting system.

CPR Rate Advisor controls

SAM 450P Trainer remote control enables the training instructor to play “Push faster,” “Push slower,” “Good speed” or “Begin CPR” during CPR compressions.

Pause button

Temporarily stops, then resumes, the training scenario to enable the trainer to offer additional explanation and support to the class.

Volume configurable

Four selectable volume levels with default to last selection.

Rechargeable battery

Operates up to seven hours—a full day of training—on a single charge. Recharges in just six hours.

Battery status indicator

LEDs indicate charging status (while not in use) or one hour of remaining battery life (while in use).

Upgradable

Updates to Trainer software available for download on the HeartSine website.

Language options

Language options enable the Trainer to provide instructions in the local language.



Specifications

Weight: 1.3 lb (0.6 kg)

Battery: Rechargeable

HeartSine samaritan PAD Trainer models

HeartSine samaritan PAD Trainers have no capability to diagnose or treat an arrhythmia. They are intended to simulate the use of an AED for instructional purposes only.



HeartSine samaritan PAD 450P Trainer with CPR Rate Advisor simulation
(TRN-450-XX*)



HeartSine samaritan PAD 360P Trainer
(TRN-360-XX*)



HeartSine samaritan PAD 350P Trainer
(TRN-350-XX*)

*XX represents two-character language code.

Accessories



HeartSine samaritan PAD Trainer remote

SAM 450P (TRN-ACC-19)

SAM 360P (TRN-ACC-18)

SAM 350P (TRN-ACC-16)



Trainer-Pak

(replacement electrode cartridge)
(TRN-PAK-04)



Replacement electrodes

(for use with Trainer-Pak)

10/pak (TRN-ACC-02)

25/pak (TRN-ACC-03)



Battery charger

(replacement)

(TRN-ACC-14)



USB data cable

(PAD-ACC-02)

For further information, please contact your Stryker representative or visit our website at strykeremergencycare.com

Emergency Care Public Access

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: CPR Rate Advisor, HeartSine, Pad-Pak, Pediatric-Pak, samaritan, Saver EVO, Stryker, Trainer-Pak. All other trademarks are trademarks of their respective owners or holders.

Date of Issue: 04/2023

Made in U.K.

H009-036-306-AG EN-US

HeartSine SAM 450P is not available for sale outside of the U.S. or Japan.

Copyright © 2023 Stryker.



Manufactured by:

HeartSine Technologies, Ltd.
207 Airport Road West
Belfast
Northern Ireland
BT3 9ED
United Kingdom
Tel +44 28 9093 9400
Fax +44 28 9093 9401
heartsinesupport@stryker.com
heartsine.com

Distributed in U.S. by:

Stryker Emergency Care
11811 Willows Road NE
Redmond, WA, 98052 U.S.A.
Toll free 800 442 1142
strykeremergencycare.com