



HeartSine®

Lifesaving technology designed for the real world

Date: January 2009
H017-014-029-0

US Technical Memo

Changes to samaritan® PAD Trainer Pak (TRN-PAK-01) and Charger.

The following technical statement has been issued by HeartSine Technologies to ensure all users of HeartSine products are aware of changes to the PAD Trainer Pak (TRN-PAK-01) and Charger.

Beginning with Lot 141 of the PAD Trainer Pak TRN-PAK-01, changes were made to the components used in its construction. These changes involved a new rechargeable battery and required charger.

PLEASE NOTE:

The new charger has 1 red LED and a two pin plug attached.

The older charger has 1 Green LED, 1 Yellow LED and 1 x yellow button with adaptors for the required plug.

The new charger may be used to charge all lot numbers of the PAD Trainer Pak.

WARNING: Using the old charger with new batteries will cause damage to the PAD Trainer Pak.

To establish the Lot number of your PAD Trainer Pak check the label which can be found inside by opening the PAD Trainer Pak. To safely recharge the HeartSine samaritan® PAD Trainer follow the instructions below:

- Insert the charger adaptor into the socket on the PAD Trainer Pak. Ensure the adaptor is fully inserted.
- Plug the charger into the electricity supply. Where required, an adaptor will have been provided with your charger.
- Check that the red light on the charger is on. This indicates that the PAD Trainer Pak is charging.
- Charge times
 - Approximately 12 hours to fully charge a completely depleted PAD Trainer Pak (lot 141 and later versions) with the new charger.
 - Approximately 20 hours to fully charge a completely depleted pre Lot 141 PAD Trainer Pak with the new charger.
- Remove the adaptor from the PAD Trainer Pak. Unplug the charger and store it.
- Turn on the PAD Trainer to ensure that the device is operating.

HeartSine Technologies
Technical Support
215-860-8100-ext 107