


samaritan® PAD 500P Trainer

Training Defibrillator with Remote Control

Provides Realistic Training Demonstration and Superior Retention

The easy-to-use samaritan PAD Trainer with remote control guides users through simulated analysis, energy delivery and prompted Cardio Pulmonary Resuscitation (CPR) intervals with the look and feel of a live samaritan Public Access Defibrillator (PAD) 500P without the actual charge and discharge of an electrical shock.

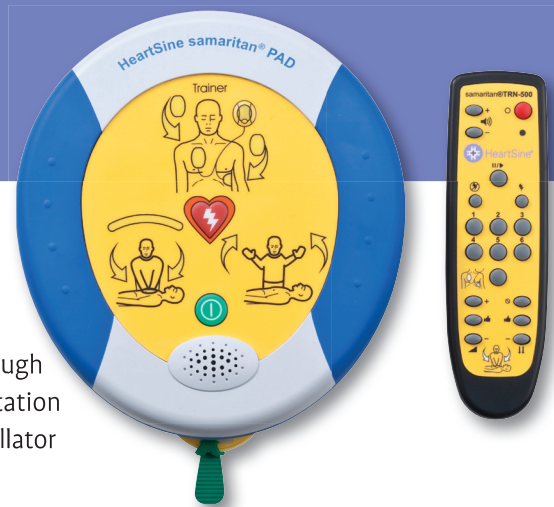

10/31/13

HeartSine UK
500P Trainer flyer
FRONT

- Finish size A4 8.26 x 11.69
- CMYK
- 4/4
- 300 dpi
- Bleeds two sides
- Stock and quantity TBD

Simulate the Full Range of “Real Life” Scenarios

The multi-function remote control enables you to select among six AHA/ERC scenarios for live demonstrations or training. One remote control can even operate multiple trainer units at a time (within a range of 3-5 metres/10-15 feet). The Trainer is also compatible with any CPR manikin system.

Realistic

Voice and visual prompts and CPR Advisor match those of the HeartSine samaritan PAD defibrillator. Likewise, size and shape are the same. Unique labeling and membrane color prevent inadvertent use in an actual emergency.

Appropriate for All Skill Levels

The Trainer can be used by businesses to train employee first responders on the use of the HeartSine samaritan PAD. In addition, it meets the needs of professional trainers, either teaching first-time lay users or refreshing experienced emergency responders.

Cost-Effective

The Trainer is powered by a rechargeable battery. Electrode pads are cost-effective and reusable, and mimic the Pad-Pak™ battery/pad cartridge unique to the HeartSine samaritan PAD defibrillator.



The Whole Package. The HeartSine samaritan PAD Trainer package includes remote control with AAA batteries, Trainer-Pak, battery charger, soft carry case with two zipper compartments, user manual, data cable and five pairs of training electrode pads.

Scenarios

The HeartSine samaritan PAD Trainer integrates realistic rescue scenarios for AED training and adheres to the AHA/ERC 2010 protocol of one shock followed by two minutes of CPR. A metronome and flashing prompt at 110 beats per minute help to establish proper rate for chest compressions.

The selectable training scenarios include:

- ▶ SHOCK ADVISED (SA) condition, followed by CPR
- ▶ NO SHOCK ADVISED (NSA) condition, followed by CPR
- ▶ CHECK PADS, followed by an SA condition and CPR
- ▶ Starts directly in “ANALYZING” mode, followed by SA & CPR, SA & CPR, and NSA & CPR
- ▶ SHOCK ADVISED (SA) condition, followed by CPR, 30 compressions and 2 breaths
- ▶ NO SHOCK ADVISED (NSA) condition, followed by CPR, 30 compressions and 2 breaths



Product Features and Details



Features

Cable rewind mechanism. Easy replacement of pads and cables after each session reduces time spent resetting system. ▶NEW

Rechargeable Battery. Operates up to seven hours—a full day of training—on a single charge. Recharges in just four hours. ▶IMPROVED

Battery status indicator. LEDs indicate charging status (while not in use) or one hour of remaining battery life (while in use). ▶NEW.

Pause Button. Temporarily stops, then resumes, the training scenario to enable the trainer to offer additional explanation and support to the class.

Volume Configurable. Four selectable volume levels with default to last selection. ▶IMPROVED

Selectable Training Scenarios. Six pre-programmed AHA/ERC scenarios. ▶IMPROVED

Training Pads. Five pairs of reusable, replaceable training electrode pads included. Compatible with all training manikins, the new electrode pads are now easier to place back on the pad liner. ▶IMPROVED

Upgradable. Updates to Trainer software available on the HeartSine website.

Language Options. Twenty-nine language options enable the Trainer to provide instructions in the local language.



10/31/13

HeartSine UK
500P Trainer flyer
BACK

- Finish size A4 8.26 x 11.69
- CMYK
- 4/4
- 300 dpi
- Bleeds two sides
- Stock and quantity TBD

Product Details (TRN-300-xx)

Specifications

- ▶ Weight: 0.6 kgs (1.3 lbs)
- ▶ Battery: Rechargeable
- ▶ Includes: Remote control with AAA batteries, Trainer-Pak, battery charger, soft carry case with two zipper compartments, user manual, data cable and five pairs of training electrode pads

Replacement Accessories

- ▶ Remote replacement (TRN-ACC-17)
- ▶ Data cable (PAD-ACC-02)
- ▶ Pad-Pak electrode cartridge (TRN-PAK-04)
- ▶ Battery charger (TRN-ACC-14)
- ▶ Electrodes: 10/pak (TRN-ACC-02), 25/pak (TRN-ACC-03)



Lifesaving, Pure and Simple

Authorised Distributor

U.S./Americas

HeartSine Technologies, Inc.
121 Friends Lane, Suite 400
Newtown, PA. 18940
Toll Free: (866) 478 7463
Tel: (215) 860 8100
Fax: (215) 860 8192
info@heartsine.com

Europe/Rest of the World

HeartSine Technologies, Inc.
203 Airport Road West
Belfast, Northern Ireland BT3 9ED
Tel: +44 28 9093 9400
Fax: +44 28 9093 9401
info@heartsine.co.uk



The products described in this brochure all meet the applicable European Medical Directive requirements.

© 2013 HeartSine Technologies, Inc.
All rights reserved.H009-032-216-0



HeartSine®
Inventor. Innovator. Lifesaver.

www.heartsine.com