HeartSine samaritan® PAD 500P Upgrader (3.4.0)

User instructions

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Windows XP
Windows Vista
Windows 7
Windows 8
Windows 8.1

HeartSine samaritan® PAD 500P Upgrader (3.4.0)

User instructions

IMPORTANT: Do not use the PAD Universal Upgrader for this upgrade.

1. Install PAD 500P Upgrader.

EITHER: Download from website:

Go to <http://heartsine.com/500P/system-updates/> & click the "HeartSine Samaritan PAD 500P Upgrader (3.4.0)" link shown opposite.

Following download, run "PAD500PUprader_setup.exe" to install PAD 500P Upgrader.
(Follow on-screen instructions.)

OR: Install from CD:

Insert CD into the computer's CD-ROM drive.
After a few seconds, the 'AutoPlay' window should appear*.
Click "Run PAD500PUprader_setup.exe" to install PAD 500P Upgrader.
(Follow on-screen instructions.)
*(If this does not happen, browse to the CD-ROM and double-click "PAD500PUprader_setup.exe")

PAD 500P System Update

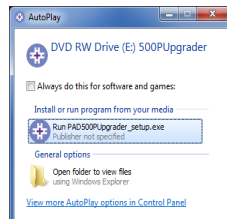
PAD 500P Upgrader

(Updated 09/03/07, UT, 3.4.0)

You have been directed to this webpage to enable you to upgrade your PAD500P device in accordance with HeartSine Technologies' field safety corrective action dated 09/03/07.

PAD 500P Upgrader User Instructions (PDF, combined English, Deutsch, Español, Français, Italiano)

HeartSine Samaritan PAD 500P Upgrader (3.4.0)

2. Start PAD 500P Upgrader.

Start PAD 500P Upgrader by clicking on the icon created on your desktop, or press Start menu button and select PAD 500P Upgrader.

After a few seconds the 'PAD 500P Upgrader - (3.4.0)' window will appear.


3. Attach PAD 500P to computer's USB port (see User Manual).

After a few seconds, the samaritan® PAD icon will appear and the connection indicator will turn green.

Click the 'Program PAD' button to begin the upgrade.

Follow any on-screen instructions.

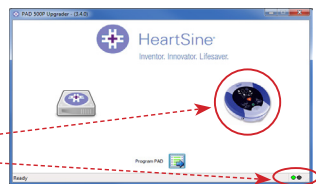samaritan® PAD icon
connection indicator


4. Following Upgrade.

After successful completion of the device upgrade and successful device on/off check, ensure that the status indicator continues to flash green approximately once every five seconds.

The device can now be returned to use.

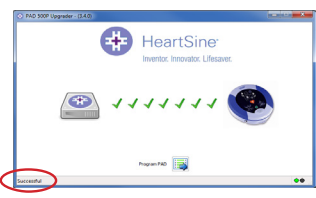If any error messages are heard or if the LED is flashing red you **MUST** contact your distributor or contact support@heartsine.com.

5. Return Proof of Upgrade.

If you are unable to return the PDF Upgrade Certificate via the web link, please return this in one of the following ways:

1. E-mail to support@heartsine.com
2. Print out and post to HeartSine Technologies
3. Fax to +44 (0)28 9093 9401

6. Upgrade Issues.

If an error occurs at any point during the upgrade, close the message box and please restart at Step 2.

Important Warning

A small number of users have reported a difficulty with performing the software upgrade. **If you encounter any such issues please contact HeartSine immediately.**

If you have any queries, please contact your distributor or e-mail support@heartsine.com

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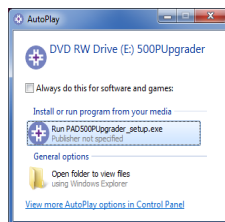
PAD 500P Upgrader

(Updated 09/04/09, 01, 3.01.0)

You have been directed to this webpage to enable you to upgrade your PAD500P device in accordance with HeartSine Technologies' field safety corrective action dated 09/04/09.

[PAD 500P Upgrader User Instructions](#) (PDF, combined English, Deutsch, Español, Français, Italiano)

[HeartSine Samaritan PAD 500P Upgrader \(3.4.0\)](#)

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HeartSine Technologies Inc
HeartSine Technologies Ltd